



SECOND BARRIER
FACE SHIELDS

Covid-19 Risks Demand a Second Barrier Between The Virus & You



This face shield acts as a second layer of protection and should be used in conjunction with your typical PPE. A second barrier is meant to reduce contact with your face from particulates, droplets, and touch.

Please see step by step assembly instructions on the reverse side.

The Second Barrier Face Shield was developed in response to the unprecedented challenges posed by the COVID-19 pandemic. It's easy to assemble and versatile for use in various occupations with exposure concerns. It can be easily disinfected with appropriate wipes after use.

To reorder please visit secondbarrier.com

Health Canada Approved Class 1 Medical Device MDEL 11941

This product is not an industrial use shield and should not be used in place of traditional PPE.




The Second Barrier Face Shield is manufactured by Sleek Advertising LTD. which has retooled its production facility to meet the needs of the COVID-19 pandemic. Please contact sales@secondbarrier.com for any questions or concerns.

ASSEMBLY INSTRUCTIONS


Step One:

Remove the protective film from both sides of all parts.

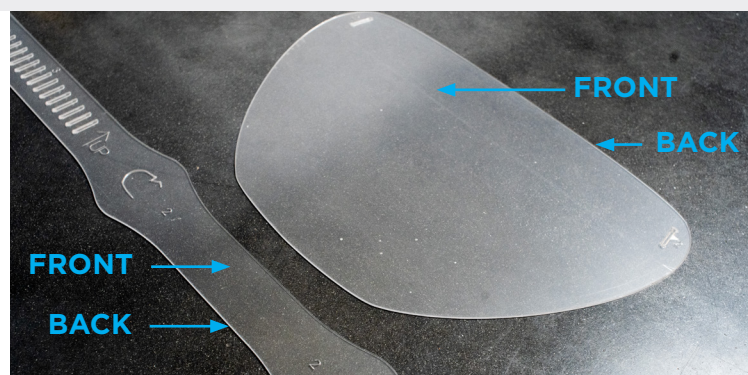

Step Two:

Be sure the arrow is pointing up and insert Tabs 2 into Slots 2 on the face shield.

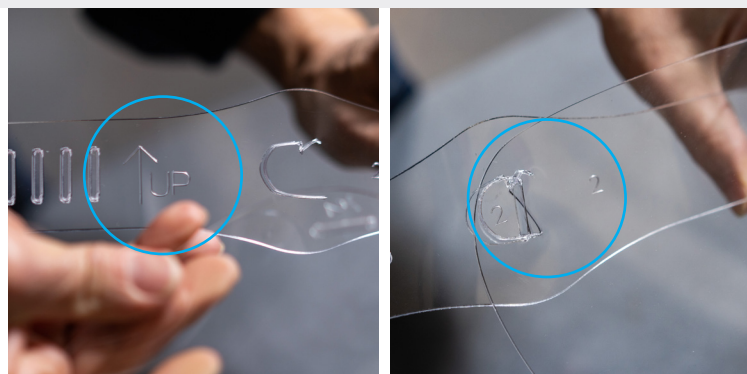

Step Three:

Fit the strap around your head and feel which slot fits best. From the inside, insert Tab 3 into the Slot 3 that feels most comfortable and isn't too tight.



Step Four:

Insert Tab 4 into Slot 4 to secure.



No claims, representations or warranties, whether express or implied, are made by our companies as to the safety, reliability, durability and performance of any of our companies' products. The Second Barrier Face Shield is to be used as additional Second Barrier over existing PPE. Not as a primary source. Our company accepts no liability whatsoever for the safety, reliability, durability and performance of any of our companies' products.

SECONDBARRIER.COM